

NAF
FIRST AID



NaturalintTMX
Veterinary approved poultice



**Directions
For Use:**

NAF NaturalintX Poultice can be applied in three different ways:



**Hot Poultice
Dressing:**

Cut the Poultice to size and place in a shallow tray of boiled water, which has been allowed to cool to hand hot temperature. Once saturated, remove the dressing and gently squeeze out excess water. Place the poultice on the affected area with the plastic backing away from the skin. Hold in place with a NaturalintX Wrap.



**Cold Poultice
Dressing:**

First soak the poultice in boiled water, then allow to cool thoroughly, before applying to the affected area. The poultice can also be chilled further, by placing it in its outer plastic bag in a refrigerator.



**Dry Poultice
Dressing:**

Simply apply the poultice to the affected area directly. Ensure the poultice is applied with the plastic away from the skin before holding in place with the NaturalintX Wrap.

Caution:

Clean the area thoroughly before the application of NaturalintX Poultice.

Clean hands thoroughly before and after application.

Do not apply the NaturalintX Wrap or bandage too tightly, as circulation can be severely restricted.

NAF NaturalintX Veterinary approved poultice should be renewed at least every 12 hours and should not be used in conjunction with any medical preparations unless advised otherwise by a veterinary surgeon.

Prior to applying the poultice to your horse, ensure that the temperature is suitable on your own skin.

Both extremes of hot and cold can be dangerous to your horse.

**Active
Ingredients:**

Each poultice contains: Tragacanth 6g, Boric Acid 2g.

TO AVOID DANGER OF SUFFOCATION, KEEP PACKAGING OUT OF REACH OF BABIES AND CHILDREN.

FOR ANIMAL USE ONLY. FOR EXTERNAL USE ONLY.



Batch No: See Side Product code: 149



Reg. No. GB 662 0001

ISO 9001
Quality Assured Firm

For further information on
using this product contact:
NAF, Wonastow Rd,
Ind Est West, Monmouth, NP25 5JA
call our Freephone Advice Line: 0800 373 106
or email info@naf-uk.com V2

